Gutta Percha Points

Puntas de Gutapercha

Pointes de Gutta Percha

Guttapercha-Spitzen

Punte di Guttaperca

Pontas Guta-Percha

 **CB Healthcare Consulting GmbH**

(manufacturer)

Am Neumarkt 34

22041 Hamburg, Germany

www.supreme-endo.com

Rev.2023/06

Symbols / Símbolos / Symboles / Symbole / Simboli / Símbolos

Use by date / Utilizar por fecha / Utiliser par date /
Haltbarkeitsdatum / Usare entro la data / Usar por data

LOT

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EN

Hand-Rolled Endodontic Points

Used to fill a prepared root canal prior to tooth restoration.

Composition

Gutta Percha, Zinc Oxide, Barium Sulfate, Coloring Agent

Application

1. Check the product if it is damaged, presence of impurities, and shape is suitable for use.
2. Check if the product is within the expiration date.
3. After treating the root canal of the damaged tooth, use the proper size gutta percha to fill.
4. This product is disposable and discarded after use.

Intended Purpose

1. Intended Use: Used to fill a prepared root canal prior to tooth restoration.
2. Indication: Used to fill a prepared root canal prior to tooth restoration.
3. Contraindication: Do not use for patient who is allergic to this product.
4. Patient Population: All age group
5. Intended user: Dentist
6. Side Effects/Adverse Effects: There are no side effects or adverse effects reported.
7. Potential risk if re-used: Re-use of used Gutta percha point may cause inflammation.
8. Can be used with sealer or cement and it shall be used under the control of the dentist according to the clinical indication.
9. Single Use Device

Precaution

1. Do not use other than dental specialist.
2. Do not use if the product is expired.
3. Before use, check for infection inside the root canal.
4. Do not use for patient who is allergic to this product.

Storage

Store at room temperature away from direct sunlight

Disposal

Dispose of contents/container according to the related regulations of the country.

Shelf Life

4 years from the date of manufacture

Radiopaque

Equivalent to 6mm aluminium stepwedge

According to the EU Medical Devices Regulation, users / patients are obliged to report serious events with a medical device to the manufacturer and to the competent authority of the country in which they occurred.

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